# User instructions

DE	Benutzeranleitung	4
EN	User instructions	6
ES	Guía de usuario	8
FI	Käyttöohje	10
FR	Mode d'emploi	12
HI	उपयोगकर्ता गाइड	14
IT	Guida utente	16
JA	マニュアル	18
KO	사용 설명서	20
NL	Gebruiksaanwijzing	22
PT	Guia de usuario	24
RU	Руководство пользователя	26
SV	Bruksanvisning	28
TR	Kullanıcı talimatları	30
ZH	使用说明	32

# UINO

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## **EN • UNO is a multipurpose splint and traction device for supporting upper and lower limbs in emergency medical situations.**

### **REMOVAL, STORAGE AND DISPOSAL**

The UNO splint can be removed by hand. Store the device in a dry place under 40°C, avoiding direct sunlight. The product can be discarded to burnable waste.

### **WARNINGS AND PRECAUTIONS**

- The device is intended for use by professionals or trained personnel. Make sure that the product is suitable for treating the patient's injury. Check contraindications before using the product.
- Do not fasten the splint too tightly to block blood circulation
- Only use for temporary supporting, and monitor limb vitality when using the device
- Use the traction unit only temporarily. If treatment is prolonged, use a designated traction device.
- Neurovascular function should always be assessed both before and after applying the UNO splint and during transport.
- Make sure that the application you are using is sufficiently strong for the indication in question.

## **CONTRAINDICATIONS**

There are no absolute contraindications for using the UNO splint in the emergency setting. Relative contraindications are:

### **Open fractures**

Special caution is needed when choosing to splint an open fracture. Reduction must be done carefully to avoid causing further damage to soft tissues, especially neurovascular structures. Massive hemorrhage should be stopped, and the wound should be protected from dirt and further contamination before applying the splint.

### **Massive trauma or unstable patient**

If the patient is severely injured or unstable, no time should be wasted to splint fractured extremities. The patient should be brought immediately to a trauma hospital according to local protocol.

### **Peripheral vascular disease, neuropathy, or injury to neurovascular structures**

Patients with loss of sensation and/or compromised vascular supply in their extremities due to either prior disease or current trauma should be splinted with extra caution due to the higher risk for pressure sores and vascular compromise.

### **Critical bleeding**

As described in the ATLS protocol, a critical bleeding must be handled before splinting a fracture. If needed, a tourniquet must be applied before applying the UNO splint. If a massive hemorrhage ensues after splinting (e.g., gunshot on evacuation), the UNO splint must be removed before applying a tourniquet to control the bleeding.

### **Impending compartment syndrome**

If the patient has symptoms of impending compartment syndrome, the decision to splint must be taken with care. A tight splint could potentially increase the compartment pressure and can hinder timely and proper evaluation of an impending compartment syndrome.

## **NOTICE TO USERS**

Severe adverse events must always be reported to the product manufacturer and the national supervisory authority. The instructions listed above are only intended to be used as a basic reference.

Local guidelines/protocols should be adhered even if in conflict with the above stated.

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© 2022 Dassiet Oy